


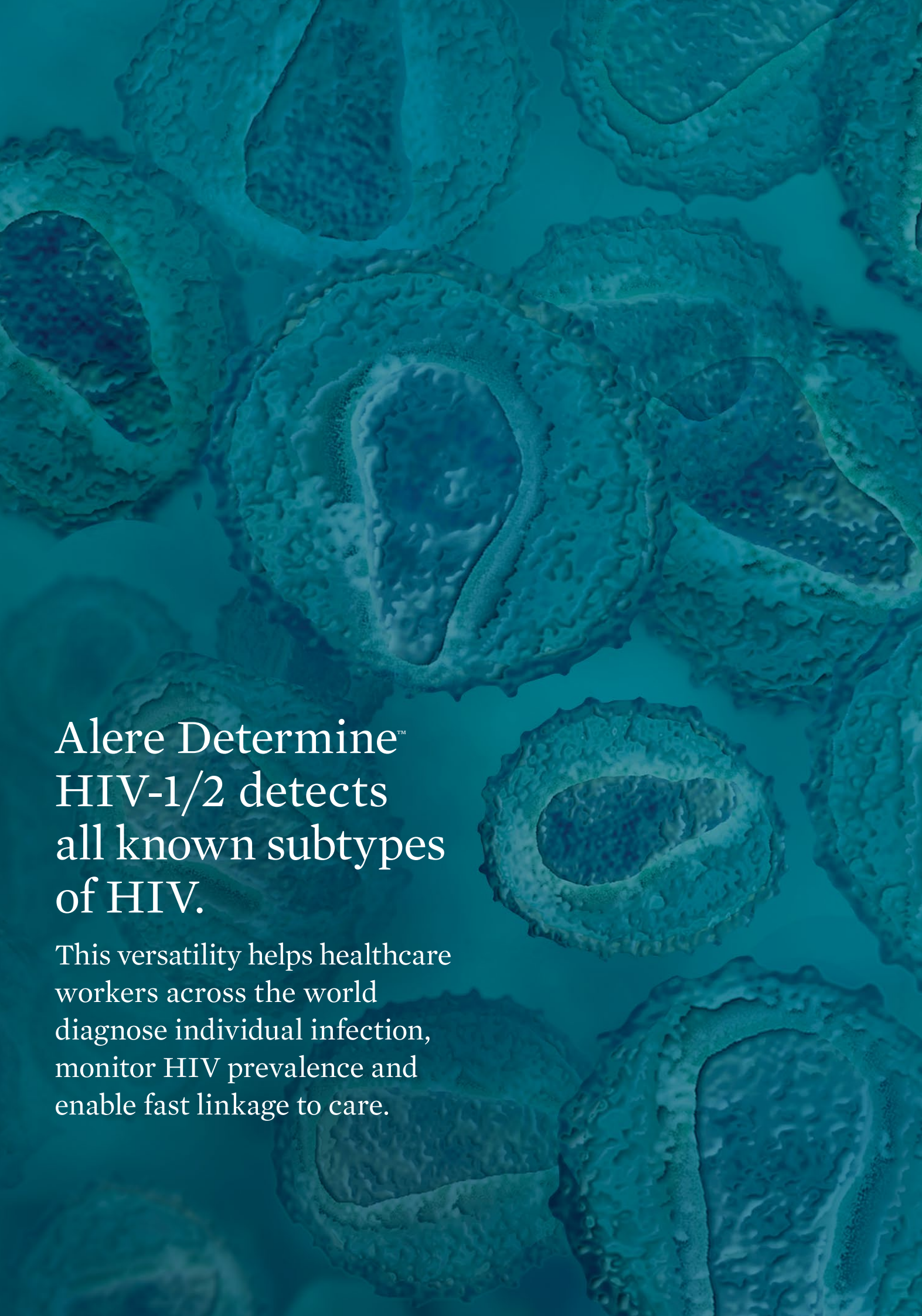

ALERE DETERMINETM HIV-1/2 TEST

Rapid HIV screening, tested
by billions, proven by practice.

A microscopic view of several HIV virus particles, showing their characteristic spherical shape and textured surface. The particles are set against a dark blue background.

Alere Determine[™] HIV-1/2 detects all known subtypes of HIV.

This versatility helps healthcare workers across the world diagnose individual infection, monitor HIV prevalence and enable fast linkage to care.



In 2017, there were 36.9 million people living with HIV.

In that year, 1.8 million new infections occurred and 940,000 people died.¹ HIV, or human immunodeficiency virus, is the virus that causes AIDS (acquired immuno-deficiency syndrome).

HIV prevention and control programs all over the world have made the Alere Determine™ HIV-1/2 antibody test their number one choice for HIV screening.

THE GLOBAL CHOICE

The test's proven field reliability truly makes it a trusted partner in the battle against HIV. Over 1.4 billion tests have been used worldwide, enabling billions of people to know their status.

Rapid HIV screening is a foundation of Abbott's commitment to providing advanced, affordable and accessible technology that is helping to enhance patient management at the point of care.

By providing tools to help diagnose patients with HIV and monitor their condition, Abbott is putting patients on the path to better health.

SIMPLY SOPHISTICATED FOR FAST AND EASY RESULTS

With its simple one-step procedure for serum/plasma or two-step procedure for whole blood, the Alere Determine™ HIV-1/2 test is quick and easy to use, delivering clear, dependable results in just 15 minutes.

Designed with simplicity in mind, minimal training is required to get started and no additional equipment, electricity or running water is needed.



UNRIVALLED SUPPORT AND SUPPLY


Every week millions of tests leave the manufacturing facility in Japan on their journey to HIV programs throughout the world. Abbott's expansive and robust production capacity ensures no compromise in product supply, so your tests are there when you need them.

This capacity also provides the flexibility to rapidly respond to large and unforecasted customer demands, so testing can continue without disruption.

Precision manufacturing has been refined over more than a decade of production experience, making the Alere Determine™ HIV-1/2 one of the most assured and reliable rapid HIV tests in the world.

The test's enduring presence in global HIV control and prevention programs has created an expansive network of trained health care workers; a confident and experienced workforce that can consistently produce reliable results.


And Abbott's global footprint provides unparalleled reach for customer support, so help is always at hand.

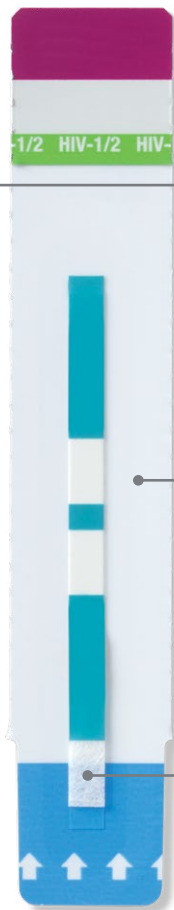

**EASY-PEEL DESICOVER
PROTECTS TEST**



**CLEAR DEPENDABLE
RESULTS IN JUST 15 MINS**

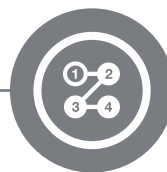

**SIMPLE PROCEDURE,
EASY TO USE**

QUALITY COUNTS

We know that quality counts and we know that quality isn't easy. Manufactured in Japan, the Alere Determine™ HIV-1/2 test is produced under the highest quality standards. For our manufacturing team, quality is a way of life and a commitment to provide the products you deserve, every time.

COMPACT AND FLEXIBLE FOR STORAGE AND SHIPPING

Designed with a wide temperature range, providing you with flexible storage options to suit your environment.

Clever packaging design means easier storage and shipping. Small and lightweight, the Alere Determine™ HIV-1/2 test allows you to maximize your valuable storage space. The removal of unnecessary packaging materials greatly reduces shipping costs while also producing less waste, for a better impact on the environment.




100 TESTS = 106g




EASY TO STACK MULTIPLE KITS FOR COMPACT STORAGE

PROCEDURE


- 1 Prepare Test**
Tear one strip from the right and remove cover.

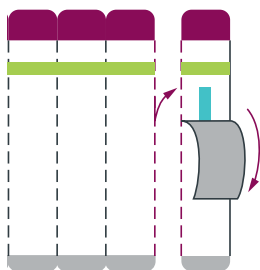


- 2 Add Sample Plasma or Serum**
Add 50 µL of serum or plasma to the sample pad and wait 15 minutes.

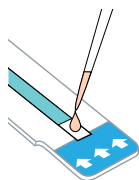



- Add Sample Fingerstick Whole Blood**
Add 50 µL of whole blood to the Sample Pad. When all the blood is absorbed into the Sample Pad, immediately apply one drop of Chase Buffer to the Sample Pad.

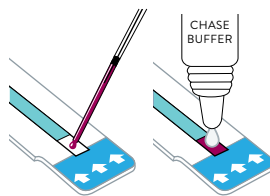

- Add Sample Venipuncture Whole Blood**
Add 50 µL of whole blood to the Sample Pad. Wait 1 minute and add one drop of Chase Buffer.

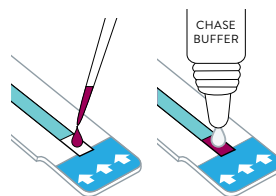

- 3 Read Result**
Read the results at 15 mins to 60 mins maximum

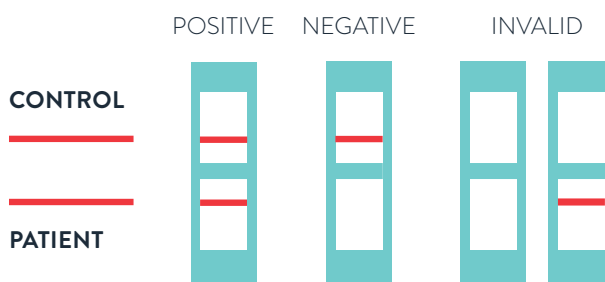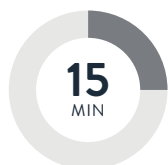

PRODUCT INFORMATION

INFORMATION TYPE	PRODUCT DETAIL
Method	Lateral flow
Time to results	15 mins.
Storage conditions	2 - 30°C
Shelf life	18 months
Sample type	Serum/Plasma/ Whole blood
Kit contents	20 or 100 test units, Package Insert

The Alere Determine™ HIV-1/2 test is pre-qualified by the WHO and is on the USAID List of Approved HIV/AIDS Rapid Test Kits.

ORDERING INFORMATION

PRODUCT NAME	CE MARKED	NON CE MARKED
Alere Determine™ HIV-1/2 (x20)	7D2346	7D2342
Alere Determine™ HIV-1/2 (x100)	7D2347	7D2343
Chase Buffer*	7D2243	7D2243
EDTA-Capillary Tubes (x100)	7D2227	7D2222

* For whole blood, order Chase Buffer Product not available in the US. Product exclusively available for export to select markets

1. UNAIDS Fact Sheet 2018

**CONTACT YOUR LOCAL ABBOTT REPRESENTATIVE TODAY,
OR VISIT ABBOTT.COM/POCT**

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. IDEM166364 12000866-02 07/19

